

# QUICK TEST RESULTS

The dog is DEA1.1 +



Presence of red line in front of "DEA1.1" arrow indicates a **positive reaction**.

The dog is DEA1.1 -



Presence of red line in front of "DEA1.1" arrow indicates a **negative reaction**.

The cat is A group



Presence of red line in front of "A" arrow indicates a **positive reaction for A group** and a negative for B group.

The cat is B group



Presence of red line in front of "B" arrow indicates a **positive reaction for B group** and a negative for A group.

The cat is AB group



Presence of 2 red lines in front of "A" and "B" arrow indicates a **positive reaction for both A and B group**.

## VERY IMPORTANT FOR ALL QUICK TEST :

The control must always show a positive reaction in front of the "C" arrow, ensuring the test has run successfully.


If not the test must be repeated.

# BLOOD GROUP COMPATIBILITY

In **dogs**, the determination of DEA 1.1 antigen is strongly recommended before any blood transfusion to avoid a potent alloantibody response against this antigen and to avoid acute haemolytic transfusion reaction.



In **cats**, the presence of naturally occurring alloantibodies in type A and in type B cats requires that blood typing be performed prior to blood transfusion to avoid acute haemolytic transfusion reaction, and in breedings to prevent neonatal isoerythrolysis.

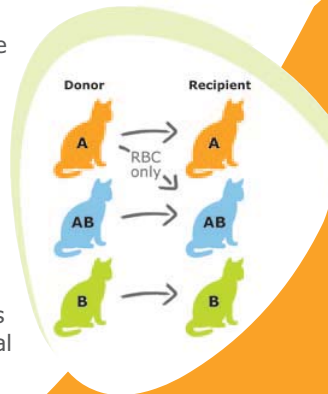

# INSTRUCTIONS FOR THE QUICK TEST PROCEDURE



To Order Alvedia Quick Test Blood Typing Kits:



Phone: (800) 824-3703  
Online: [www.mwivet.com](http://www.mwivet.com)



Phone: (888) 722-2242  
Online: [www.AAHAMARKETLink.com](http://www.AAHAMARKETLink.com)

**Alvedia**  
Alice Veterinary Diagnostic

